



**Klausz Dental
Laboratories Ltd.**

Working **harder and smarter** for your practice!

www.klauszdentallab.com • klauszdl@uniserve.com • Tel: 416-630-6262 • 1-800-410-4392
40 Kodiak Crescent • Units 11-12 • Toronto, ON • Canada M3J 3G5

An O.I.C.S. Specialty Laboratory *Occlusion - Implants - Cosmetic- Sleep*

Low Cost, Simple to Use Sleep Apnea Device

The myTap is a simple, patient friendly treatment for snoring and sleep apnea. The appliance advances and stabilizes the mandible in an easily controlled, protrusive position so that the soft tissues of the throat and tongue do not collapse into the airway, causing snoring and sleep apnea.

The myTap Concept

The trays of the myTap look like teeth whitening trays and are made from a unique plastic, that when heated become soft and can be molded to fit over the upper and lower teeth. They couple together to hold the mandible in a forward position so that the jaw does not fall back during the night. The more the jaw is advanced, the more the airway can open.

How It Works


The myTap adjustment mechanism is attached to the lower tray and is threaded through the adjustment post of the upper tray. An adjustment dial is screwed onto the end of the threaded adjustment mechanism allowing the doctor or patient to incrementally adjust the advancement of the mandible.

The Advantages Are Clear

- Low cost
- Easy to fit
- Small and sleek
- Quick turnaround time
- Easy to titrate (protrusion/retrusion)
- No special bite registration is required
- Adjustable while in the patient's mouth
- Can be easily remolded to accommodate other treatments



Get to Know the Key Components

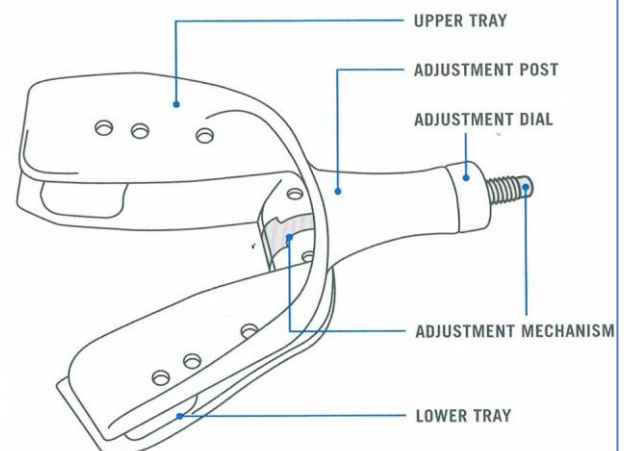

Every day we strive to exceed your expectations and being able to provide you with another great product is just another way we are, "Working Harder and Smarter for your Practice!"