

INSTRUCTIONS FOR USE



**O2VENT
OPTIMA**
TM
**ORAL APPLIANCE
FOR SLEEP APNOEA
THERAPY**



OVENTUS[®]

OADS^{Ltd.}
It's All About The AIRWAY

Name of User

Name of Sleep Physician

Name of Dentist

Serial No. of Device

Date of first fitting/purchase

Connector band starting titration length

Date of follow up with Physician/Dentist

INSTRUCTIONS FOR USE FOR O₂VENT OPTIMA™

CONTENTS

Introduction	2
Optima Description	3
Your Optima kit	4
How Your Optima Device Works	5
Using your Optima	8
Inserting your Optima	8
Removing your Optima	8
Finding the right titration setting	9
What are the Risks of OSA	11
Contraindications	11
Cleaning Instructions	12
General Warnings and Precautions	14
Maintenance	15
Replacing connector bands	16
Keep a Log of your Titration	17

INTRODUCTION

Congratulations, you have taken the first steps to improve your sleep and well-being!

Your Healthcare Professional has chosen the O₂Vent Optima™ (**Optima**) specifically for your snoring and/or obstructive sleep apnoea (OSA) therapy. OSA is a condition where a sufferer experiences repetitive episodes of partial or complete blockage of the upper-airway during sleep. This is caused by obstructions of the nose, soft palate and tongue.

The **Optima** is a removable oral appliance that is fitted in the mouth and is intended to reduce or alleviate snoring and mild to moderate OSA. The **Optima** is indicated for use during sleep to aid in the treatment of these conditions. Intended for adult patients 18 years or older. The **Optima** device is for prescription use only.


OPTIMA DESCRIPTION

Your **Optima** features the following components:



Optima parts (before assembly):

lower tray, upper tray and connector bands

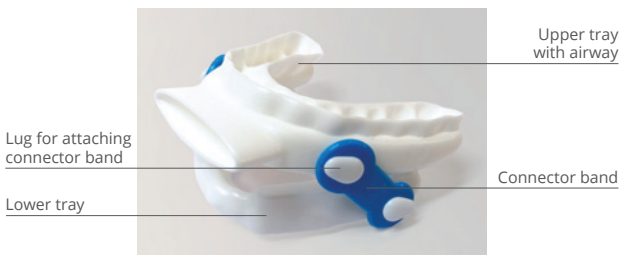

Optima (fully assembled)

YOUR *OPTIMA* KIT

Your customised *Optima* comes fully assembled with 19mm length connector bands*. The *Optima* kit contains:

- Two sets of connector bands to adjust titration. Each set of bands has 9 length size options ranging from 13-21mm. The size is marked on the band.
- Oventus tub
- Instructions for use
- Warranty
- Antibacterial cleanser tablets

It is important that you check the contents of your personalised *Optima* kit and make sure that you find everything listed above. Please contact your Healthcare Professional if anything is missing.

* In some instances, your Healthcare Professional may recommend a different starting length for your connector bands based on your jaw anatomy.



HOW YOUR *OPTIMA* DEVICE WORKS

In general, the *Optima* works in the following way.



Brings/pulls the lower jaw forward (this opens the airway and stabilises the tongue)


Provides a unique air channel in case of nasal congestion or difficulties with breathing through the nose.

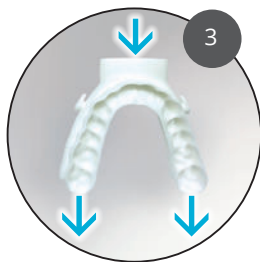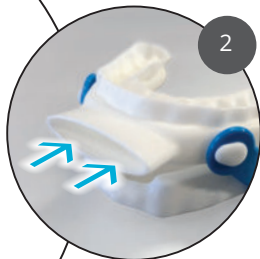

The air channel allows the air to flow smoothly to the back of the airway.

HOW YOUR *OPTIMA* DEVICE WORKS CONTINUED

THE *OPTIMA* DEVICE

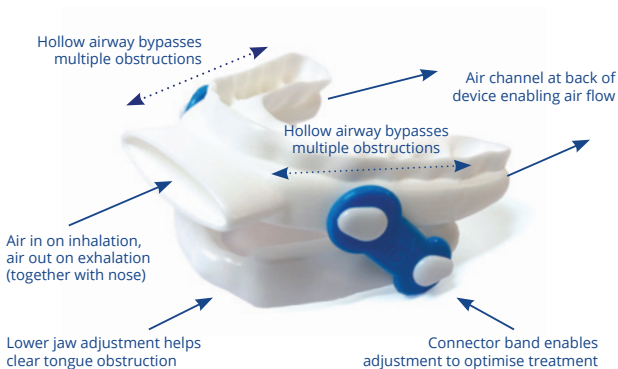

In order to manufacture the *Optima*, an image of the teeth using 3D scanning or impressions are taken. The information is sent to a laboratory where your customised *Optima* will be made. When the *Optima* is ready, your Healthcare Professional will fit and assess how well the *Optima* works to treat your snoring and/or OSA.

The **Optima** consists of the following parts:

- Upper tray (upper teeth)
- Lower tray (lower teeth)
- Connector bands

The upper and lower trays are connected to each other by the connector bands and will be delivered assembled:

- The bands are used to move the lower jaw (i.e. adjust the trays) forward or backward.
- In order to adjust the trays to your specific needs, the bands come in different lengths to enable titration adjustment. There are 9 length size options ranging from 13-21mm.
- Your new **Optima** is delivered fully assembled with the 19mm connector band as the recommended starting titration length for all new users*.
- The shorter the connector band, the further the lower jaw is moved forward.
- After the initial use, your Healthcare Professional will help you to find the right length of connector bands to receive the best therapy.

* In some instances, your Healthcare Professional may recommend a different starting length for your connector bands based on your jaw anatomy.

USING YOUR *OPTIMA*

INSERTING YOUR *OPTIMA*

BEFORE YOUR SLEEP PERIOD

1. Inspect your *Optima*; make sure the air channel is clear.
2. Make sure your *Optima* is connected with the bands and they are the same length on each side.
3. Make sure your *Optima* is clean.
4. Rinse the *Optima* and brush your teeth before inserting.
5. Using your thumbs, insert the upper tray and gently “push” onto the upper teeth; the tray should fit snugly.
6. Insert the lower tray, “push” onto the lower teeth; the tray should fit snugly.

REMOVING YOUR *OPTIMA*

AFTER YOUR SLEEP PERIOD

7. Remove your *Optima* by gently opening your mouth, lift up the lower tray and then pull down on the upper tray. You might feel a little “resistance” initially.
8. Make sure **BOTH** trays are loosened before removing.



The *Optima* trays should never be worn separately. Always wear both trays (assembled with the connector bands) when using the device.

FINDING THE RIGHT TITRATION SETTING

- Your **Optima** will be delivered ready to use, fitted with the 19mm connector bands as the recommended starting titration for all new users*.
- During the **initial** phase of using your **Optima**, you will work closely with your Healthcare Professional to find the correct position. ** It is important that you wear your **Optima** each sleep period according to your Healthcare Professional recommendations**
- Ensure the connector band lengths are the same on each side.
- The titration may take several weeks or longer to achieve the best therapy.
- If your Healthcare Professional suggests a shorter or longer length band, refer to the diagram on page 10 for length and titration information.
- After your settings have been established, you should wear your **Optima** each sleep period.

* In some instances, your Healthcare Professional may recommend a different starting length for your connector bands based on your jaw anatomy.


USING YOUR *OPTIMA* CONTINUED

The table below shows the lower jaw positioning range that can be set by the connector band lengths. This is a guide only.

The recommended starting titration for new users in most instances is the 19mm connector bands.

NOTE: The shorter the connector band, the further the lower jaw is advanced. The longer the connector band, the less the lower jaw is advanced.

OPTIMA LOWER JAW POSITIONING RANGE

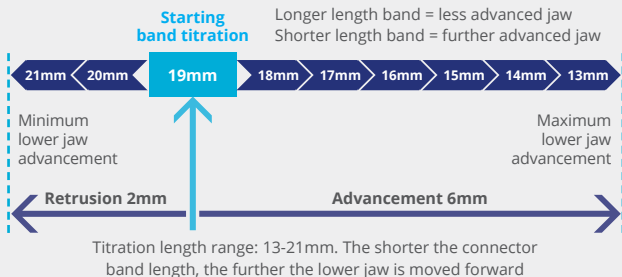

TIPS

- If you are having problems adapting, try wearing your *Optima* while watching TV or reading. It will help you to get used to using it.
- It might be helpful for you to track how much you use your *Optima*, this will help you to get into the habit of applying your *Optima* before your sleep period.

WHAT ARE THE RISKS OF OSA?

- Detrimental effects on sleep-quality and health
- Increases the risk of obesity
- High blood pressure
- Stroke
- Heart attack
- Type-2 diabetes
- Depression, impotence, mood disorders and increased risk of motor vehicle and workplace accidents

It is important to use your **Optima** during each sleep period to receive the best possible outcomes.

CONTRA-INDICATIONS

The **Optima** device is contraindicated for patients who:

- Have central sleep apnoea
- Have severe respiratory disorders
- Have loose teeth or advanced periodontal disease or restorative needs
- Are under 18 years of age

OPTIMA CLEANING INSTRUCTIONS

RECOMMENDED AFTER EACH USE

- Rinse the **Optima** with lukewarm water and clean with a soft clean toothbrush (not the same toothbrush you use to brush your teeth, as toothpaste can damage the device).
- Thoroughly rinse the device with lukewarm water, including through the airway entry and exit points.
- Ensure the airway entry and exit are free from any debris.
- Dry your device after each cleaning with a clean tissue or towel and shake to clear water from the airway.
- Return to the Oventus tub to store, with the lid open to allow the internal airway to dry.



RECOMMENDED TWICE A WEEK

- Clean your **Optima** device more thoroughly twice a week, or more frequently as required, using the provided Oventus tub and an antibacterial cleanser (free from chlorine, bleach, acetic acid or alcohol-based products).
- Conduct the daily cleaning as above.
- Fill the Oventus tub provided about $\frac{3}{4}$ full (about 10mm below the top) with warm water.



- Place the antibacterial cleanser and the **Optima** device in the water so that they are completely submerged and leave them to soak, following the recommended time according to the cleanser packet instructions.
- Remove and rinse both the **Optima** device and Oventus tub thoroughly with lukewarm water, including through the airway entry and exit points, to remove the cleaning agent.
- Dry your device after each cleaning with a clean cloth.
- Return to the Oventus tub to store, with the lid open to allow the internal airway to dry.
- The **Optima** may change colour over time, which could be due to certain foods/liquids. This will not change the effectiveness of the device. If you use coloured mouthwash after cleaning your teeth, wait 15 minutes before inserting your **Optima**, to avoid discolouration. For best results, follow these provided cleaning instructions.



GENERAL WARNINGS AND PRECAUTIONS

Use of the **Optima** may cause the following side effects:

- Tooth movement, discomfort, pain or changes in dental occlusion
- Gum irritation, gingival or dental soreness
- Pain or soreness to the temporomandibular joint (TMJ)
- Dry mouth or excessive salivation
- Cheek or tongue pain
- Jaw discomfort, pain or jaw set (clenched jaw)
- Temporary bite change. This will subside approximately 30 minutes after the **Optima** is taken out of the mouth after each sleep period
- Permanent bite change may occur as a result of long term use

If any side effects or ongoing dental issues persist while using the **Optima**, please seek advice from your Healthcare Professional.

PRECAUTIONS:

- Never use chlorine, bleach, acetic acid or alcohol-based products (including mouthwash) to clean your **Optima**
- Only use warm water, not to exceed 45°C or 113°F (boiling water may cause damage)
- Do not use toothpaste or mouthwash to clean your **Optima**
- If you require dental work such as fillings, crowns, extractions etc., these may change the fit of your **Optima**. You should always discuss this with your Healthcare Professional
- The **Optima** trays should never be worn separately

MAINTENANCE

Your **Optima** should be stored in the Oventus tub provided as shown, during transport or when not in use.



Keep your device away from sunlight and out of reach of pets and children.

Once you have determined the band length that optimises your therapy, you can re-order them in packs of 25. To re-order the connector bands, ask your Health Professional or visit www.ventus.com.au


The **Optima** connector bands have a recommended usage time frame of one month, as usage can affect the band length and the titration of your device. The connector bands should be replaced every month or more frequently if required.

REPLACING CONNECTOR BANDS

The connector bands are essential for keeping your **Optima** device assembled, with the lower jaw set at the chosen position eg. with 19mm bands. The connector bands need to be replaced every month as they may lose elasticity. To identify when both bands need changing, compare their length against a new band. If the bands are longer or the holes stretch from circular to oval, it is time to change both bands.

To remove a connector band from your **Optima** device, pull the upper and lower device pieces apart (as shown in diagram). Peel the band away from the wider end of the “lug” and off the “lug” point.

To apply a connector band, hook the band over the pointed end of the “lug” and pull gently to hook over the wider “lug” end.

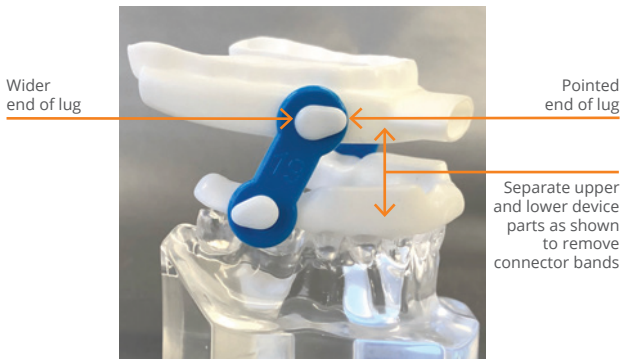

WARRANTY:

Please see the Oventus Warranty Policy Card included with your device.



MANUFACTURER :

Open Airway Dental Solutions Australia Pty Ltd.

5 Moorak Street,

Taringa, QLD 4068, Australia

Phone: +61 1300 533 159

Email: info@openairway.com

MANUFACTURED FOR:

Open Airway Dental Solutions Ltd.

301 - 295 College Street

Toronto, Ontario, M5T 1S2, Canada

Phone: 1-877-977-8267 (Toll Free)

Email: info@openairway.ca



OVENTUS[®]

OADS_{Ltd.}

It's All About The AIRWAY